

lead to an increased risk of infection due to cross contamination, and the properties of the device may no longer be optimal for its intended use.

3. For external use only and as directed.
4. Do **NOT** use past expiry date or if the product is damaged.
5. The product is sterile and must **NOT** be used if the packaging is damaged or open.
6. See side of sachet for embossed Expiry/Lot information.
7. Keep out of reach of children.
8. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

Disposal:

After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.



DO NOT USE
IF PACKAGE
IS DAMAGED

DO NOT
RE-USE



DO NOT
RE-STERILISE



KEEP
DRY



KEEP AWAY
FROM
SUNLIGHT

ALCOHOL, FRAGRANCE, LATEX,
PARABENS AND PHTHALATES FREE

MEDICAL DEVICE



DANGER. Highly Flammable.

Keep away from heat, hot surfaces, sparks, open flames, and other ignition sources. No smoking. Use in a well-ventilated area. Store away from direct sunlight. Store at ambient temperature and humidity.

FOLD

FOLD



Manufacturer:



Medicareplus
INTERNATIONAL Est. 2008

Medicareplus International Limited.
Chemilines House, Alperton Lane,
London, HA0 1DX, United Kingdom.
Telephone: +44 (0)20 8810 8811
www.medicareplus.co.uk

Authorized Representative in the
European Community:



Advena Ltd. Tower Business Centre, 2nd Flr.,
Tower Street, Swatar, BKR 4013 Malta

Date of Issue: 7 June 2021.

IFU0003/ver.2



STERILE

Total Barrier Film Applicator Non-Sting

INSTRUCTIONS FOR USE

CE
2797



Total Barrier Protection Strategy

Total Barrier Protection is a complete cost-effective treatment strategy against Moisture-Associated Skin Damage (MASD) and offers a range of products adapted to the location and severity of the skin damage (mild, moderate, severe).

The range consists of **MEDI DERMA-S** Total Barrier Cream, **MEDI DERMA-S** Total Barrier Film, **MEDI DERMA-PRO** Foam & Spray Incontinence Cleanser, **MEDI DERMA-PRO** Skin Protectant Ointment, and **LIFTEEZ** Medical Adhesive Remover.

Intended Use:

MEDI DERMA-S Total Barrier Film

Applicator is a sterile device designed to provide a transparent, quick drying and long-lasting film protection (up to 72h) from Moisture-Associated Skin Damage (MASD) on intact and moderately damaged skin. Its application helps to prevent irritation and skin breakdown from incontinence, other bodily fluids (sweat, wound exudate, stoma leakage), adhesive products and friction. Ideal for skin folds, periwound, peristomal and intravenous site protection.

Information for Use:

MEDI DERMA-S Sterile Total Barrier Film Applicators are supplied in two sizes, 1ml and 3ml. The product is sterile and must **NOT** be used if the packaging is damaged or open.

Benefits:

- Long-lasting protection from Moisture-Associated Skin Damage (MASD)
- Suitable for application to intact and moderately damaged skin on adults and paediatrics
- Non-Sting formulation, pain-free application

FOLD

- Alcohol, fragrance, latex, parabens, and phthalates free
- Not made with natural rubber latex
- Quick drying (5-10 seconds)
- Does not affect the adhesion of the ostomy pouch, adhesive device or dressing
- Prevents skin irritation and breakdown
- Prevents exudate damage and maceration
- Protects the area where the ostomy pouch, adhesive device or dressing is to be applied
- Minimises the risk of shear and friction-related trauma

Ingredients:

Siloxanes.

Directions for Use:

1. Ensure skin is clean and dry prior to application.
2. Apply a uniform coating over the whole area to be protected, or where the ostomy pouch, adhesive device or dressing will be applied.
3. Wait for 5-10 seconds for the barrier film to fully dry.

FOLD

4. If you have missed a particular area, please wait until the initial application has dried, then reapply as required.
5. If applicable, once fully dry, apply the ostomy pouch, adhesive device or dressing as per manufacturer's instructions.
6. Reapplication is recommended every 24-72 hours depending on the frequency of cleansing and severity of incontinence. When used under adhesive dressings or devices, reapply at each dressing or device change.

Contraindications:

1. Do **NOT** use on open wounds or deep puncture wounds.
2. Do **NOT** use on infected areas of skin.
3. Do **NOT** use where there are any known allergies or sensitivity to the ingredients.

Precautions:

1. Consult your clinician should any signs of skin reaction occur following product application.
2. This device is for single-use only and should not be re-used. Re-use may

Ordering Information:

Description	Product Code	PIP Code
MEDI DERMA-S Sterile Total Barrier Film Applicator - 1ml/5 Pack	61076	362-8716
MEDI DERMA-S Sterile Total Barrier Film Applicator - 3ml/5 Pack	61090	362-8724

Available on Prescription from your GP or Healthcare Professional and is supplied via Pharmacy or your normal delivery service.