

AVAILABLE IN AEROSOL, SPRAY, WIPE AND APPLICATOR

USER
GUIDE

MEDI DERMA-S

TOTAL BARRIER FILM

TOTAL
BARRIER
PROTECTION™

 Medicareplus
INTERNATIONAL



How to Use MEDI DERMA-S Total Barrier Film



Step 1: Skin Preparation:

- Clean and dry the skin.




Step 2: Application:

- Spray using an even sweeping motion holding the can 5cms away from the skin (pump spray) and 10-15cms (aerosol). Allow 30 seconds to dry.



Step 3: Application:

- If using an applicator or wipe, apply evenly to the affected area. If using around a wound or stoma, apply to the surrounding skin. Allow 5-10 seconds to dry.

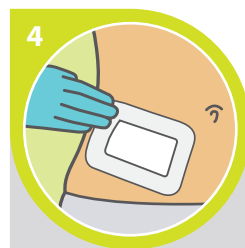

Step 4: Apply Dressing/Device:

- Let the film dry before applying a dressing, adhesive device or ostomy pouch (if using). Reapply film as instructed.



Total Barrier Film:

For **EXTERNAL** use only. Do **NOT** use on infected areas of skin. Do **NOT** use if there are signs of skin reaction following product application, or if there are known allergies/sensitivities to any of the ingredients. Consult your clinician accordingly. Only use as directed. Do **NOT** use past expiry date or if product is damaged. For single patient use only. Keep out of reach of children.

PART OF OUR
SKIN PROTECTANTS
& WOUND
CARE RANGE



SCAN HERE For more
information on **MEDI DERMA-S**
Total Barrier Film



Medicareplus
INTERNATIONAL

Medicareplus International Limited

Chemilines House | Alperton Lane | London | HA0 1DX | UK

Telephone: +44 (0)20 8810 8811 | Email: info@medicareplus.co.uk

www.medicareplus.co.uk

Follow us on:



@medicareplusuk

LT305PT-F021023V1