

MEDI  
DERMA-S

# FACIAL SKIN PROTECTION FOR FFP3 MASKS OR EQUIVALENTS

Our **MEDI DERMA-S Total Barrier Film** is effective in cases where irritation behind masks is caused by perspiration and friction, when used in line with our guidance.

We **DO NOT** advocate the use of our barrier film products for the prevention of damage that may be caused solely as a result of direct pressure.


## Prevention and Maintenance Normal Intact, Red/Discoloured Skin

### MEDI DERMA-S Total Barrier Film Wipe and 1ml Sterile Applicator

- Ensure the skin is clean and dry before applying the barrier film
- Using the Wipe or Sterile Applicator, apply the film in a uniform coating to the area of the face to be protected; **ensuring the eyes are kept closed during application**
- Allow **at least 30 seconds** for the film to dry after application before applying the mask
- Apply mask as normal, following any instructions provided
- **Film application should be repeated at the start of every shift**

## Superficial Face Wounds and Broken Skin

**MEDI DERMA-S Total Barrier Film** may be used underneath adhesive dressings to prevent further skin damage from repeated removal and reapplication. *(Please follow your local advice/guidance on the use of dressings under masks).*

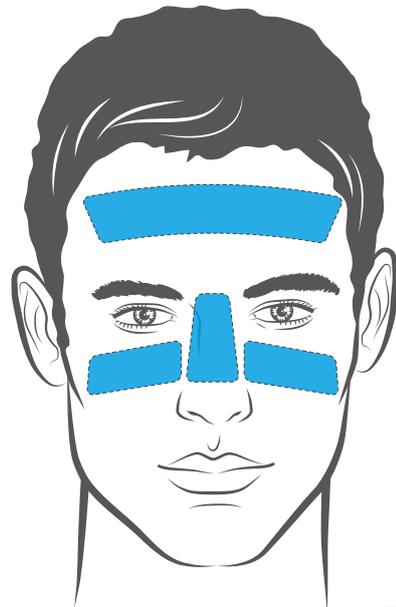

## Ordering Information

Product	Format	Selling Unit	Product Code	PIP Code	NHS-SC Code
<b>MEDI DERMA-S</b> Total Barrier Film	Wipe	30 Pack	60307	341-3184	ELY788
	Sterile Applicator	1ml - 5 Pack	61076	362-8716	ELY532